INSTRUCTIONS FOR USE

ISOVAC®

Dental Isolation Adapter

EN

DA

DE

EL

ES

FI

FR

IT

JA

NL

NO

PL

PT

SV

TR

TABLE OF CONTENTS
INDHOLDSFORTEGNELSE
INHALTSVERZEICHNIS
ΠΙΝΑΚΑΣ ΠΕΡΙΕΧΟΜΕΝΩΝ
CONTENIDO
SISÄLLYSLUETTELO
TABLE DES MATIÈRES
INDICE
目次
INHOUDSOPGAVE
INNHOLDSFORTEGNELSE
SPIS TREŚCI
ÍNDICE
INNEHÅLLSFÖRTECKNING
İÇİNDEKİLER

ISOVAC®

Dental Isolation Adapter

Isovac® Dental Isolation Adapter: Instructions for Use

EN

Isovac® tandisolationsadapter: Brugsanvisning

DA

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[REF](#) EIV0101

[REF](#) EIV0102

This system is intended for use with the single-use sterile or non-sterile Mouthpieces developed by Zyris, Inc. These Mouthpieces are available in a wide range of sizes for effective isolation.

Each Isovac Dental Isolation Adapter is comprised of one main component: Control Head

Control Head

[REF](#) PIV0101

The Control Head includes two Vacuum Levers, a Right (P/N: 20-0405) and a Left (P/N: 20-0406), to adjust suction in the upper and lower areas of the Mouthpiece.

Suction Line Plug

[REF](#) PIL0206

O-ring Maintenance Kit

[REF](#) AIL0226

Double End Tube Brush

[REF](#) AIL0225

Lever Set

[REF](#) PIV0102

Sold separately

Mouthpiece Lubricant

[REF](#) AIL0202

Sold separately

Hose Connectors

[REF](#) AIL0227

[REF](#) AIL0234

[REF](#) AIL0231

[REF](#) AIL0235

[REF](#) AIL0232

[REF](#) AIL0236

[REF](#) AIL0233

Sold separately

Some chairs may require a hose connector to install the Isovac onto the High Volume Evacuation (HVE) valve or Saliva Ejector hose.

Barrier Sleeves

[REF](#) AIL0203 (non-sterile)

Sold Separately

Non-sterile Control Head Barrier Sleeves are available for additional protection from cross-contamination. Use of Barrier Sleeves does not eliminate the need to follow disinfecting and sterilization recommendations for the Control Head.

Mouthpieces

REF CILXX0X (non-sterile)

REF CISXX0X (sterile)

Sold separately

Sterile and non-sterile Mouthpieces are single use only and available in various sizes. The Mouthpieces are constructed of a soft, flexible polymer and are not made with natural rubber latex. They do not contain phthalates or BPA's. For more information on sizes, go to Zyris.com.

Intended Use

Isovac is a dental isolation adapter for clinical use by dental professionals. The adapter is designed to isolate two quadrants at once, simultaneously providing tongue and cheek retraction and continuous suction. The retraction and continuous suction are the important functions that control the oral environment, reducing contamination and improving bond strength while protecting the airway and soft tissue.

Contraindications

No known contraindications. Do not use without reviewing how to select the correct Mouthpiece size for each patient. Do not use if unable to position the Mouthpiece correctly.

Side Effects

Use of the Mouthpiece may trigger a pharyngeal reflex (laryngeal spasm) in hypersensitive patients, or if it is not placed correctly within the oral cavity.

How Supplied

Isovac

The Isovac Dental Isolation Adapter contains: 3 Control Heads, Suction Line Plug, Double End Tube Brush, O-ring Maintenance Kit and Instructions for Use.

Mouthpieces

Mouthpieces (sold separately) are supplied in convenient operator boxes. Each Mouthpiece is individually packaged. Non-sterile Mouthpieces are packaged inside a polyethylene bag while Sterile Mouthpieces are packaged inside a thermoform tray with a Tyvek lid. Sterile Mouthpieces are sterilized using e-beam radiation. When sterilized, yellow indicator dots on both the Tyvek lid and the box turn red. Do not use the Sterile Mouthpiece if the indicator dots are not red.

Installation

Attaching without a Connector

Isovac is designed to install easily onto an existing High Volume Evacuation (HVE) Valve or Saliva Ejector in most dental operatories. To install, detach the HVE Valve or Saliva Ejector from the tailpiece of the evacuation hose. Next, attach the Isovac directly onto the tailpiece. Press firmly in place (Figure 1).

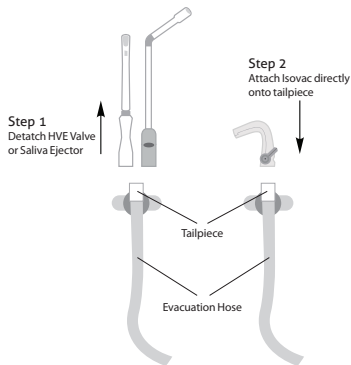
Figure 1. Installation without a Connector

Attaching with a Connector

For some dental units, a connector (sold separately) may be required to attach the Isovac to the tailpiece of the evacuation hose. To install, detach the HVE Valve or Saliva Ejector from the tailpiece of the evacuation hose. Next, attach the connector onto the tailpiece. Then, attach the Isovac onto the connector. Press firmly in place (Figure 2).

To find the correct connector, identify the connection type for your dental unit (Table 1). Contact your Customer Service Representative if your connection type is not shown.

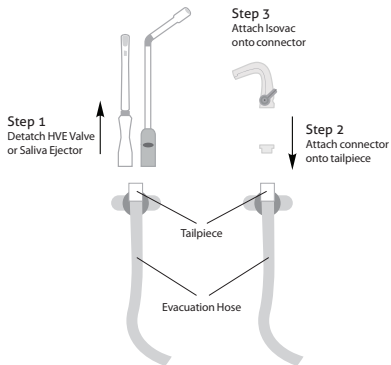
Figure 2. Installation with a Connector

Connection Type:	A-dec	Pelton & Crane Style	Sirona	Heka	XO Care	Anthon, Castellini, Finndent, Kavo, OMS, Planmeca, Stern Weber
Connector:						
Part Number:	REF AIL0227	REF AIL0231	REF AIL0233	REF AIL0234	REF AIL0235	REF AIL0236
Attaches to:	HVE	HVE	Saliva Ejector	Saliva Ejector	Saliva Ejector	Saliva Ejector

Table 1. Connection Types and Connectors

Attaching Directly onto HVE Valve / Saliva Ejector Control Heads

To attach the Isovac directly onto HVE Valve / Saliva Ejector Control Heads, use the 11 mm Connector (sold separately) (Table 2). To install, detach the vacuum tip from the Control Head. Next, attach the 11 mm Connector onto the Control Head. Then, attach the Isovac onto the 11 mm Connector. Press firmly in place (Figure 3).

Connector:	Connection Type:	Part Number	Attaches to
	11 mm Connector	REF AIL0232	HVE Valve / Saliva Ejector Control Head

Table 2. 11 mm Connector.

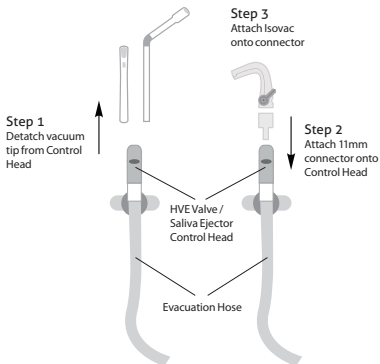
Figure 3. Installation Directly onto HVE Valve / Saliva Ejector Control Heads

How to Operate

Using the Dual Vacuum Levers

Dual Vacuum Levers adjust suction intensity in both the upper and lower quadrants. To adjust, rotate the left and right Vacuum Levers to the desired suction intensity for each quadrant. To shut off suction, rotate the levers to the most downward position.

Selecting the Mouthpiece

To select the correct Mouthpiece size, measure the patient's interincisal opening. If between sizes, you may select the smaller size.

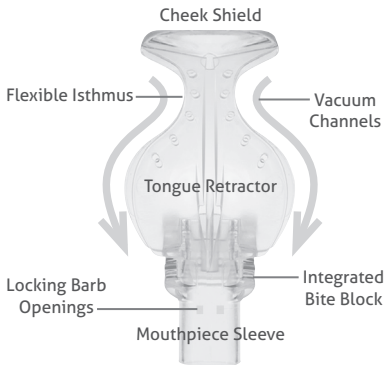
Figure 4. Mouthpiece

Mouthpiece Size	Approx. Interincisal Opening
Pediatric (PED)	< 30 mm
Extra Small (XSM)	< 30 mm
Small (SML)	30 mm
Medium (MED)	30-45 mm
Medium Deep Vestibule (MDV)	30-45 mm
Large (LRG)	> 45 mm

Table 3. Mouthpiece Sizes

The Finger Method

The finger method may also be used to select the appropriate Mouthpiece size (Figure 5).

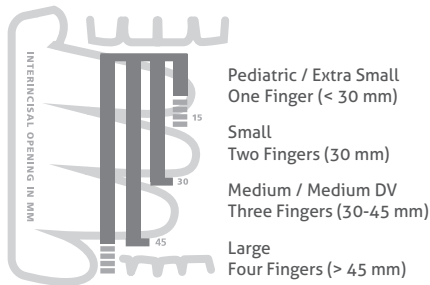
Figure 5. The Finger Method

Attaching the Sterile Mouthpiece

Follow the established protocol in your practice for handling sterile product. Open the thermoform tray by removing the Tyvek lid and place the Mouthpiece in a sterile field. Slide Mouthpiece onto the Control Head until the two locking bars engage. Sterile water may be used to lubricate the inside surface of the Mouthpiece sleeve to ease attachment.

Attaching the Non-Sterile Mouthpiece

Slide the Mouthpiece onto the Control Head until the two locking bars engage. Water or mouthpiece lubricant may be used to lubricate the inside surface of the Mouthpiece sleeve to ease attachment.

Placing the Mouthpiece

Prior to placing the Mouthpiece within the patient's oral cavity, moisten the Mouthpiece and the patient's lips.

Step 1

Hold the sides of the Control Head with your index finger and thumb. With your other hand, fold the Cheek Shield onto the Tongue Retractor. Gently slide the folded mouthpiece into the buccal vestibule on the side to receive treatment. Angle the mouthpiece to allow the lower edge of the Tongue Retractor to move along the buccal edge of the teeth.

Step 2

Move the bite block onto the occlusal surface of the teeth, just distal to the

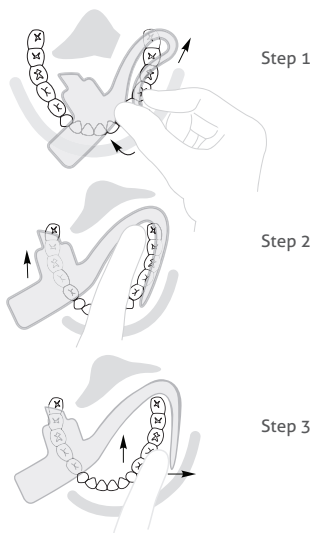
Figure 6, 7, & 8. Mouthpiece Placement

mandibular cuspid. Instruct the patient to “rest gently” on the bite block to secure.

Step 3

Place the Cheek Shield into the buccal vestibule. Move the Isthmus onto the retromolar pad behind the maxillary tuberosity. Adjust the Tongue Retractor in the lingual vestibule as needed. Move the bite block distally to provide more vertical working room.

Disassembly of the Control Head

Step 1

Holding the head of the vacuum hose, pull to remove the Control Head. Do not bend. When Isovac is removed, the Suction Line Plug (PN: PIL0206) can be used to plug the suction hose.

Step 2

Remove the Vacuum Levers by rotating the levers to “off” position and keep pushing gently until each lever pops out, or gently pry them out. (Figure 9.)

Step 3

Separate the upper and lower halves of the Control Head by pulling the two halves apart from the hinges.

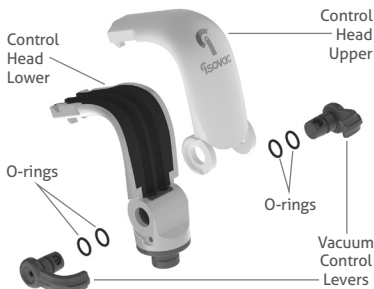
Figure 9. Isovac Dental Isolation Adapter Components

Cleaning and Sterilization

Cleaning and Sterilization of the Control Head

All components of the Isovac can be sterilized in an autoclave. Zyris recommends sterilization between each patient.

Remove biomatter and other debris from the suction channels using the provided Double End Tube Brush with a neutral pH (6.5-7.5) enzymatic cleaning solution.

After cleaning, rinse with tap water, dry, and place components in an autoclave pouch to be sterilized in a steam autoclave. Use validated sterilization cycles (Table 4).

Validated Sterilization Cycles

Cycle	Temperature	Min. Exposure Time	Min. Drying Time
Standard	132 °C / 270 °F	15 min	15 min
Prevacuum	132 °C / 270 °F	4 min	20 min
Prevacuum	134 °C / 273 °F	3 min	20 min

Table 4. Validated Sterilization Cycles

Maintenance

Control Head

After sterilization, lubricate the Vacuum Lever O-rings with the provided Dow 111 O-ring lubricant. Check the inside of the Lever for L (Left) and R (Right) for correct placement (Figure 10).

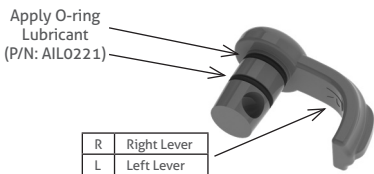
Figure 10. Vacuum Lever to Lubricate

Figure 11. Retainer Ring to Lubricate

Warnings

ⓧ **WARNING:** Mouthpieces are single use only. Reuse can result in the transmission of infectious pathogens. Mouthpieces must be discarded after use. Mouthpieces cannot be safely resterilized after use.

ⓧ **WARNING:** Do not use if the individual Mouthpiece package has been previously damaged or opened. Non-sterile Mouthpieces are individually packaged in polyethylene bags. Sterile Mouthpieces are individually packaged in a thermoform tray with a sealed Tyvek lid.

⚠️ **WARNING:** Be sure to use established protocols and patient monitoring when using the Mouthpiece in conjunction with sedation dentistry. Patient management by a certified anesthesiologist is strongly recommended.

Precautions

⚠️ **CAUTION:** Do not use the Sterile Mouthpiece if the sterilization indicator dot on the Tyvek lid is not red.

⚠️ **CAUTION:** When performing dental procedures requiring high levels of blood evacuation, it is recommended that the Mouthpiece be intermittently flushed with water to prevent coagulation in the Mouthpiece to avoid loss of suction.

⚠️ **CAUTION:** Do not autoclave the

Control Head at temperatures above 135 °C; higher temperatures could damage components and void your product warranty.

⚠ CAUTION: Failure to maintain properly lubricated O-rings can result in damage to the Isovac that is not covered by product warranty.

Limited 1-Year Warranty

Zyris warrants this product to be free from defects in material and workmanship for a period of one year from the date of original purchase. If the product exhibits such a defect, Zyris will, at its option, repair or replace it without costs for parts and labor. The product must be returned by the customer to Zyris in accordance with current Return Material Authorization procedures. Proof of date of original purchase may be required. This warranty does not cover finishes or normal wear, nor does it cover damage resulting from accident, misuse, dirt, tampering, unreasonable use, failure to provide reasonable and necessary maintenance, service performed or attempted by

unauthorized individuals or service agencies, or devices that have been otherwise modified. All implied warranties, including any implied warranty or merchantability or fitness for any particular purpose, are limited in duration to one year from the date of receipt of the product. In no event will Zyris be responsible for consequential damages resulting from the use of this product.

Instructions for Use may also be found at: Zyris.com

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All rights reserved.
© 2018 Zyris, Inc.